

sharpsafe

Smart, Safe, Sustainable



INAL SHARPS

WARNING: DO NOT FILL ABOVE THE LINE
FILL LINE LOCATED ON LID, ABOVE RIM

MADE IN
BRITAIN

6

TO BE INCINERATED
OR DEPOSED BY OTHER APPROVED METHODS, WHERE REGULATIONS PERMIT.

DO NOT OVERFILL
MAXIMUM NOMINAL VOLUME 5L
CONTAINER NOT TO BE RE-USED
DO NOT FORCE SHARPS INTO CONTAINER
MAXIMUM GROSS MASS 1.27 KG

CLINICAL WASTE,
UNSPECIFIED, N.O.S. UN 3291

BS EN ISO
23807-1:2019
KM15672

Vernacare
www.vernacare.com
Ref: 05102-0001_01

LOCKED BY
DATE
DISPOSED BY
DATE

PRODUCT CATALOGUE

Interpath Services

Leading the way

Proud Suppliers of Sharpsafe Containers



80% Recycled Plastic

Reducing virgin plastic entering the supply chain



Compliant

1st to comply to AS 23907:2023



Cost Effective

Save money while saving the world



Safe & Secure

Uncompromised protection against needle stick injuries

About Plascare

Sharpsafe is dedicated to providing solutions that protect against infections and needle stick injuries while safeguarding the environment.

As the Australasian distributor of Sharpsafe[®] and Clinisafe[®], we are at the forefront of shaping standards for safer and more sustainable sharps containers.

Sharpsafe products are trusted across a wide range of healthcare settings, including hospitals, clinics, dental practices, and is the most widely adopted container for first responders and ambulance services nationally. Sharpsafe offers ergonomic and practical solutions that are not only safer and more cost-effective but also contribute to a cleaner, safer environment.

We maintain a secure supply chain to guarantee uninterrupted service and support for our customers, promoting peace of mind and operational efficiency across diverse industries.

Pioneering Sustainability

Safe and Sustainable Waste Management

At Plascare, we align our practices with key international and Australian standards, including support for the United Nations' Sustainable Development Goals (SDGs).

This ensures we maintain transparency, accountability, and integrity in all aspects of our business. We focus on minimising environmental impact, endorsing fair labour standards, supporting community development, and maintaining a supply chain free from forced labour and human rights abuses.

These innovative containers deliver a 94% reduction in CO2 emissions compared to previous models assessed by their Life Cycle Assessment, reflecting the extensive research and collaboration with WRAP Cymru, ASTUTE at Swansea University, Bangor University and Bio Composite. We are committed to upholding the highest standards of Corporate Social Responsibility (CSR) both within Australia and globally.

We ensure our operations are sustainable, ethical, and socially responsible. Our CSR efforts encompass environmental sustainability, ethical labour practices, community engagement, and responsible governance.

CONTENTS

- 02 About Plascare
- 03 Pioneering Sustainability
- 07 Compliance
- 08 Complete Range at a Glance
- 10 Portable Sharps Disposal Containers
- 12 Small Volume Sharps Disposal Containers
- 14 Medium Volume Sharps Disposal Containers
- 15 Protected Access Lid

- 16 Large Volume Sharps Disposal Containers
- 17 Clear Lid Sharps Disposal Containers
- 18 Clinisafe Disposal Cartons
- 19 Clinisafe Disposal Bins
- 20 Brackets and Trolleys
- 22 Stands and Trays
- 23 Product Range Summary



Why Choose Sharpsafe Containers?

Sharpsafe® is a premium sharps management system designed to reduce needlestick injuries and ensure safe, compliant disposal of sharps waste. As the first to fully meet Australian Standards AS23907:2023, it sets the benchmark for safety and sustainability, trusted by healthcare facilities nationwide.



Smart

Innovative Closing

Equipped with temporary and permanent closure.

Smart Bracket System

Ergonomic accessories: Brackets and Trolleys enable safer workflow for healthcare professionals.

Space-Saving Design

Reduce storage space and transport cost.



Safe

Strength & Durability

Increased penetration resistance from 12.5 Newton to 20 Newton.

Single Hand Operation

Safely and securely open and close the container with one hand.

Protective Access

Protecting Healthcare workers and patients in busy environments.



Sustainable

Low Carbon Footprint

Low carbon Footprint made from recycled materials, reducing our carbon footprint by 94%.

Our manufacturing process is powered by solar energy.

Advancing Sustainability

Sharpsafe enables healthcare facilities to achieve greater sustainability with an eco-friendly solution.

Compliance




AS/NZS 3816:1998

Sets out requirements for the identification, segregation, handling, storage, transport, treatment and ultimate safe disposal of clinical and related wastes which may be hazardous, in an environmentally responsible manner.



ISO 23907-2:2019

The international standard for single use sharps containers. It specifies the requirements for containers intended to hold hazardous sharps medical waste including puncture resistance, drop rupture resistance, fluid tightness and correct labelling.

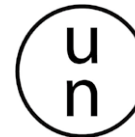

ADR Regulations

specifies the conditions that must be met by containers used when transporting infectious or associated clinical waste by land. Models of approved containers receive a UN number. All of our containers comply with these technical requirements.



FSC

Covering forestry management, FSC rules are the most comprehensive and FSC social and environmental requirements are the highest.

Complete Range at a Glance

Sharpsafe Range



Protected Access Lid



Clinisafe Cartons




Safe Clip / Adapters



Stand / Tray



Trolley / Bracket

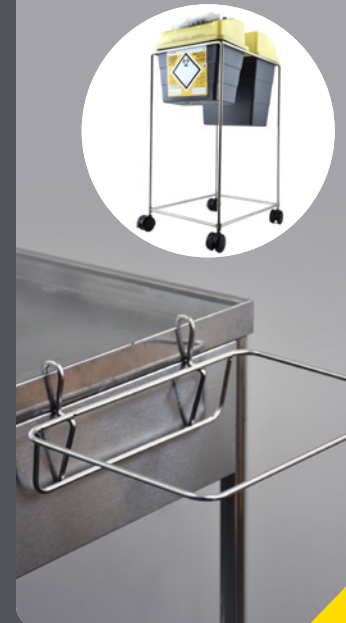

Portable Sharps Disposal Containers

0.2L / 0.45L / 0.6L

For use in first response, accident & emergency medical units, doctors and nurses bags, or for those self-administering medication. The one hand operation and extra high penetration resistance makes the portable range especially safe for transporting sharps in emergency first aid kits.

Sharpsafe portable range enables best practice in sharps disposal

Our portable range containers have the following features:

- Welded construction for added strength and safety
- Temporary and permanent closures with single handed operation
- In-mould label which cannot be removed, even by autoclaving
- Fold away handles for safe transport
- Reinforced base for increased puncture resistance
- Tested to conform to Australian, International standards

Clear Opening Petal Opening



Product Data

Product Code	5173-2430	5170-2430	5150-2439
Size	0.2L	0.45L	0.6L
Useful / Total Volume	0.17L / 0.2L	0.38L / 0.45L	0.49L / 0.6L
Container Dimensions	146 x 79 x 37	170 x 105 x 50	224 x 105 x 54
Opening Dimensions	25 x 22	25 x 10	40 x 30
Wall Bracket		✓	✓
Stand		✓	✓
Petal Opening			✓
Units per Carton	100	100	100
Carton Weight	6.4kg	9.5kg	12.75kg
Carton Volume	0.040m ³	0.081m ³	0.108m ³

What makes the Sharpsafe Portable Range superior to others?

The Sharpsafe portable range is made from up to 80% recycled polymer and is manufactured to conform with AS 23907-2023. The Sharpsafe portable range gives its users the convenience and security of having a dependable sharps disposal outlet while on the road, whether for emergency, personal or clinical use.



The Sharpsafe range of portable containers can be operated with one hand, allowing users to open and close the container with just one index finger. Users can safely control the sharps with one hand, open the container with the other and carry out disposal in the safest and most efficient manner.



Small Volume Sharps Disposal Containers

1L / 1.8L / 2L / 5L / 7L

Sharpsafe® is the number one brand when it comes to sharps containers intended for use in healthcare institutions.

Developed with sustainability and safety in mind to manage the risks of needle stick injuries. One of the key principles of sharps waste management is to bring the container as close as possible to point of care.

Principles of near patient sharps management:

- Avoid re-capping
- Mount containers onto nursing trolleys
- Stabilise on preparation tables through Sharpsafe Stands
- Bring close to patient with Sharpsafe Near Patient Sharps Trays
- Use the temporary closure and observing fill limits



Safe and Secure




Product Data

Product Code	5160-2439	5050-2430	5140-5439	5102-5430	5112-5430
Size	1L	1.8L	2L	5L	7L
Useful / Total Volume	0.85L / 1L	1.45L / 1.8L	1.7L / 2L	4.31L / 5L	5.85L / 7L
Container Dimensions	187 x 107 x 89	220 x 125 x 114	177 x 197 x 126	260 x 170 x 205	342 x 170 x 205
Opening Dimensions	58 x 36	55 x 55	50 x 55	105 x 65	105 x 65
Wall Bracket	✓	✓	✓	✓	✓
NPSD Tray	✓	✓	✓	✓	✓
Pen Driver Needle Removal	✓	✓	✓	✓	✓
Scalpel Removal	✓		✓	✓	✓
Luerlock Removal	✓		✓	✓	✓
Units per Carton	100	75	50	48	48
Carton Weight	13.7kg	12.05kg	12.1kg	16.7kg	20.4kg
Carton Volume	0.09m ³	0.089m ³	0.087m ³	0.1159m ³	0.1483m ³

Features


Quick, simple and secure assembly.	✓
2 closure positions on flip top lids: temporary and final, ensuring safety at all times.	✓
Internal safety flap ensures waste does not fall out if container is accidentally tipped.	✓
Clear visible fill line on the translucent lid to identify when container is used/full.	✓
Handle for easy and safe transport.	✓
Nestable design for compact storage	✓
Ideal for community settings or frequent use.	✓

Assembly and Use

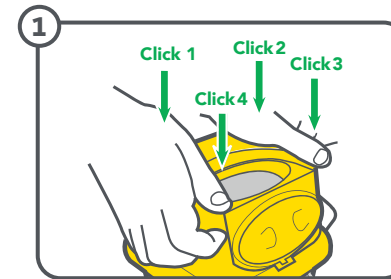

Assembly

Place the base of the container on a firm surface, then attach the lid over the base and firmly apply pressure to each corner so that it engages with a "click".

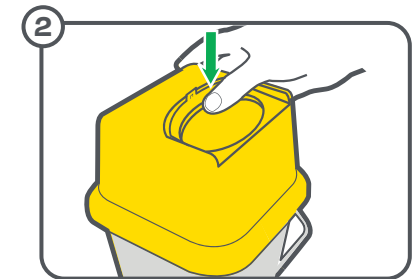


Temporary Closure

Ensure temporary closure is engaged when the container is not in use.


Safety indicators:

- Single finger operation
- 1 click heard
- Tab visually in temporary closure position

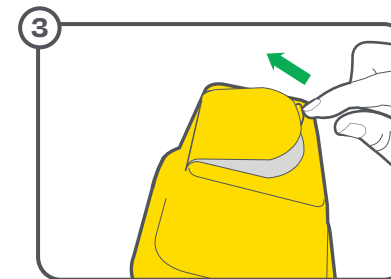

Re-Open

Push tab up to release lid from temporary closure.

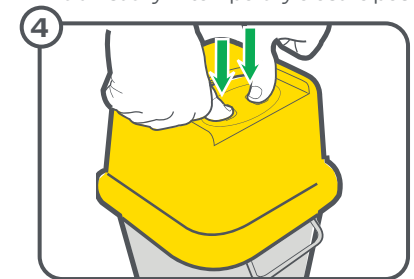

Final Closure

Place the container onto suitable surface to lock the Sharpsafe container ready for disposal.

Safety indicators:

- 2 finger operation
- 3 clicks heard
- Tab visually in permanent closure position.

Medium Volume Sharps Disposal Containers

9L / 13L / 24L / 25L

Our large volume range benefits from a large horizontal aperture to allow of easy and safe disposal sharps of all sizes. These containers can be fixed to the wall, table or on a trolley for ergonomic and safe practice.




Product Data

Product Code	5117-2430	5115-2430	5120-2439	5195-1430
Size	9L	13L	24L	25L
Useful / Total Volume	7.65L / 9L	10.5L / 13L	18.85L / 24L	21.25L / 25L
Container Dimensions	291 x 292 x 195	353 x 292 x 195	335 x 450 x 230	790 x 310 x 230
Opening Dimensions	210 x 65	210 x 65	365 x 95	260 x 165
Wall Bracket	✓	✓		
Trolley	✓	✓	✓	✓
Units per Carton	20	20	15	10
Carton Weight	13.8kg	15kg	16.5kg	16.25kg
Carton Volume	0.119m ³	0.136m ³	0.159m ³	0.138m ³

Final Closing Instructions for 9/ 13/ 24 Litre:

1. Pull flap up using central tab.
2. Rotate flap down to cover aperture.
3. Firmly apply pressure along the front of the flap.
4. Ensure flap engages with a "click".

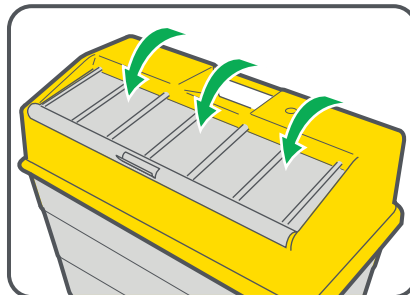

Protected Access Lid

9L PA / 13L PA / 24L PA

Protected Access allows easy disposal of sharps while providing added security for both the user & the public.

- The barrel shaped rotating aperture, provides added security for the user, patients and members of the public. The pivoting safety shield requires clear separation between hand and sharps to ensure safe disposal and further reduce the risk of needle stick injury.
- Discourages using the container for non-sharps waste, avoiding unnecessary waste disposal costs.
- Designed and intended primarily for use in public access areas (A&E, outpatients and anywhere children may be present).



Safe and Secure



Sharps Only




Suitable for Public Access




Product Data

Product Code	5117-1430	5115-1430	5120-1430
Size	9L PA	13L PA	24L PA

Disposing Instructions

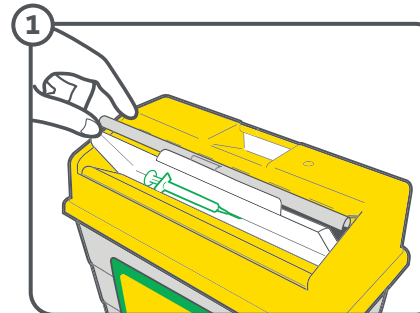

Place sharps waste in the aperture.

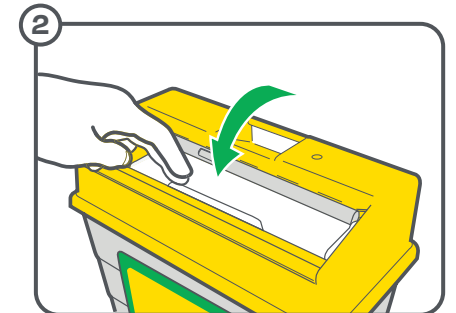

Press down on the protective access lid.

Large Volume Sharps Disposal Containers

30L / 60L Porthole

The large Clinisafe containers have been developed in accordance with the toughest safety standards. They also provide a superior level of ergonomics, sturdiness and hygiene expected by the healthcare professionals and waste collection and treatment companies.

Temporary and permanent closure states allow functional and secure workflow for all clinicians.



Safe and Secure



Fluid Tight





Temporary/Permanent Closure



Product Data		
Product Code	5185-0432	5186-0432
Size	30L	60L
Useful / Total Volume	28.5L / 30L	56L / 60L
Container Dimensions	375 x 400 x 300	670 x 400 x 300
Aperture Dimensions	153 (diameter)	153 (diameter)
Temporary and Final Closure	✓	✓
Suited to Long Objects		✓
Pedestal Trolley	✓	✓
Units per Carton	10	10
Carton Weight	16.1kg	20.9kg
Carton Volume	0.224m ³	0.214m ³

Temporary and Final Closure

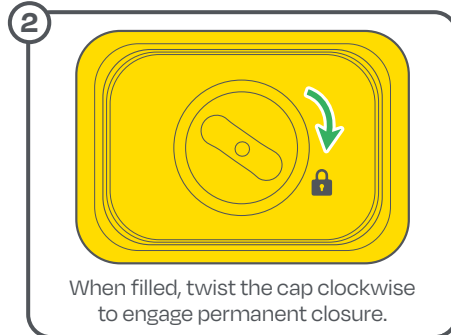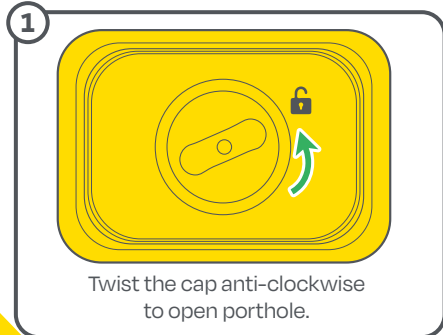

Introducing New Dual Protect Pro™



With DUAL PROTECT **PRO**

Improving safety, maximising capacity, and protecting the environment. Introducing the all new 5 & 7 Litre Sharpsafe® containers with Dual Protect Pro™.

Featuring the new Dual Protect Pro™

Designed to serve a dual purpose by protecting contents from falling out the container, and acting as a guide for optimal contents arrangement.

- **Star disconnection** for injector pens
- **Luer lock & Luer slip** combined disconnection features
- **Puncture resistant**
- **Safe assembly** – four audible clicks for secure containment
- **Clearly visible** fill lines and safe space
- **Carry handle** – compatible with existing SafeClip bracket



Greater Capacity than Competitor

The 5L & 7L are optimised in capacity. The 7L has 18% greater capacity than competitor round container. 13% Greater capacity than the previous model and 64% in-mould label.



100% Recycled Plastic Base

With 100% recycled plastic in their bases, these containers are our most sustainable yet.



Clinisafe Disposal Cartons

25L / 50L

The Clinisafe range of Cardboard Clinical Waste Containers is constructed from FSC approved recycled cardboard and contains an integral polyethylene liner with ties for closure.


The environmentally friendly, lightweight design of the Clinisafe cartons allows more containers to be delivered flat packed, reducing transportation cost and carbon emissions, as well as minimising storage space. Ideal for solid and soft clinical waste, as well as packaged liquids.

Product Data		
Product Code	2100-1001	2200-1100
Size	25L	50L
Container Dimensions	370x 264 x 264	579 x 293 x 293
Opening Dimensions	264 x 264	293 x 293
Units per Carton	10	10
Carton Weight	3.5kg	2.15kg
Pack Volume	0.0258m ³	0.041m ³

Features	
FSC approved materials	✓
UN 3291 and NFX50-507 Certified	✓
Lower emissions and transport cost	✓
Stackable when full	✓
74% Recycled Cardboard	✓
Barcoding for Traceability	✓
Flat packed design for storage	✓



Assembly and Use

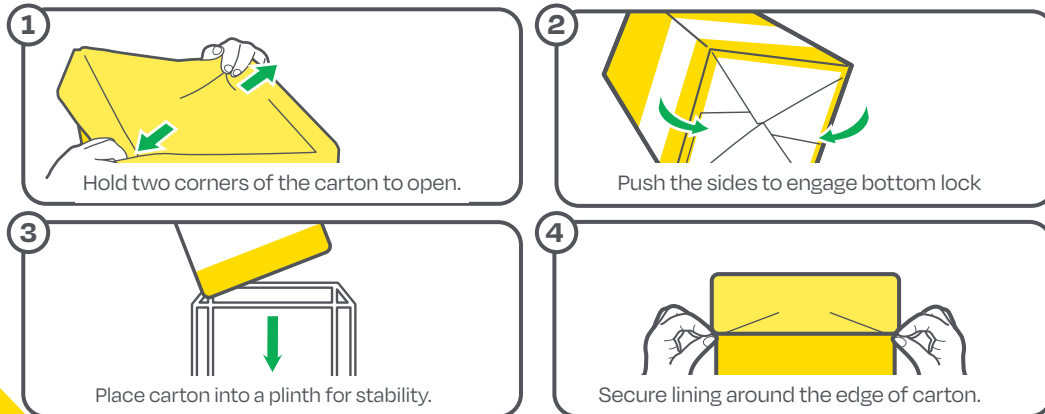

Clinisafe Disposal Bins

30L / 60L


The large Clinisafe containers have been developed in accordance with the toughest safety standards. They also provide a superior level of ergonomics, sturdiness and hygiene expected by the healthcare professionals and waste collection and treatment services.

Temporary and permanent closure states allow functional and secure workflow for all clinicians.



Product Data		
Product Code	5185-0000	5186-0000
Size	30L	60L
Useful / Total Volume	28.5L / 30L	56L / 60L
Container Dimensions	375 x 400 x 300	670 x 400 x 300
Opening Dimensions	390 x 285	390 x 285
Temporary and Final Closure	✓	✓
Suited to Long Objects		✓
Pedestal Trolley	✓	✓
Units per Carton	10	10
Carton Weight	16.1kg	20.9kg
Carton Volume	0.224m ³	0.214m ³

Assembly and Use

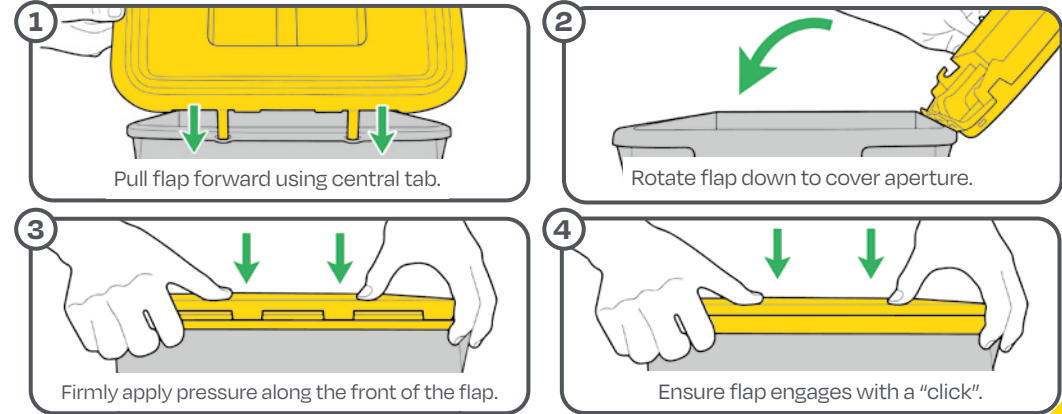

Brackets and Trolleys

1L / 1.8L / 2L / 5L / 7L

9L / 9L PA / 13L / 13L PA / 24L / 24L PA

Inadequate disposal of clinical waste is one of the leading causes of sharps injuries with 22% of needlestick injuries occurring either during transit to disposal or during the actual disposal of a sharp.

Mounts and brackets support best practice to ensure the safety of health practitioners. Sharpsafe's Near Patient Sharps Disposal system features sharps containment products that are designed to be taken directly to the patient, disposing of the needle at source. This eliminates the risk of carrying sharps to final disposal, and any potential injuries.

Sharpsafe Safe Clip – Universal wall bracket suits wide range of sizes (1L, 1.8L, 2L, 5L, 7L).

Pedestal Trolley
(9L / 13L / 24L)



Wall / Trolley Bracket
(9L / 9L PA / 13L / 13L PA / 24L / 24L PA)



Trolleys

Pedestal Trolley is lightweight and easily manoeuvrable. It allows sharps containers to be positioned at the point of use & disposal. Designed to accommodate a wide range of Sharpsafe containers and adaptors they are fitted with braking castor for added control and peace of mind.

Features

Large, easy to clean surface	✓
Easily manoeuvrable trolley frame	✓
Allows for sharps to be disposed at the point of use	✓
Safe and secure transportation	✓
Anti-static castors	✓


Features

Universal design to suit range of container sizes	✓
Made from recycled content	✓
Suitable with rubbing alcohol	✓
Retrofits to existing bracket mounts for easy change over	✓

Safe Clip
Universal Bracket (1L / 1.8L / 2L / 5L / 7L)




Assembly and Use

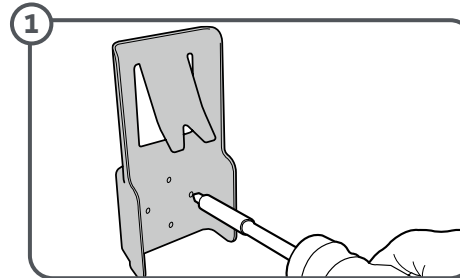


Installation

Puncture relevant screw holes and fix the SafeClip wall bracket to a wall or other suitable surface with screws provided.

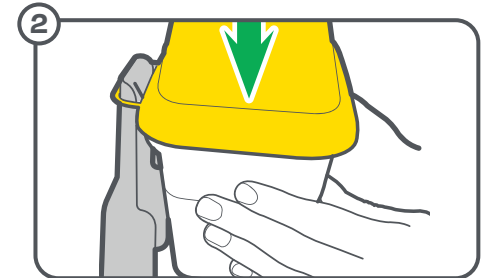

Slide and Clip

Loop the handle over the top of the bracket. Slide the handle down the bracket until the container engages the clip over the containers rim as shown.

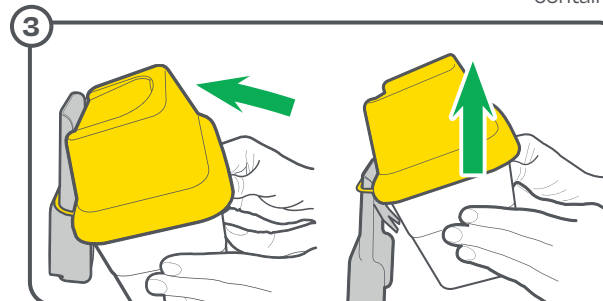

Tip to Unclip

To release the container from the SafeClip wall bracket, tip the container backwards as shown. Whilst in this position using both hands lift the container off the bracket until the handle is clear.

Stands and Trays

0.45L / 0.6L / 1L / 1.8L / 2L / 7L / 9L / 13L

Our Sharpsafe trays add extra safety and convenience to our customers, allowing sharps to be disposed of at point of use, thus minimising opportunities for needlestick injuries.

Sharpsafe Portable Stand




NPSD* Tray



*Near Patient Sharps Disposal


Accessories	Compatible with	Product Code
Sharpsafe 0.45 / 0.6 Litre Stand	0.45/ 0.6L	4151-0000
Sharpsafe 1/ 1.8/ 2 Litre Stand	1L/ 1.8L/ 2L	4205-0000
Sharpsafe 5/ 7 Litre Stand	5L/ 7L	4257-0000
Sharpsafe 9 / 13 Litre Stand	9L/ 9L PA/ 13L/ 13L PA	4207-0000
NPSD 1 / 1.8 / 2 Litre Tray	1L/ 1.8L/ 2L	4300-0000

How to use NPSD™

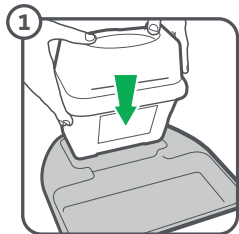


Preparation

Select the correctly sized container and tray for the procedure.

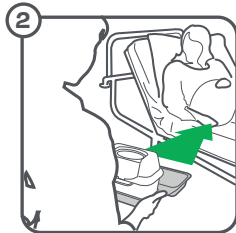


Point of Use

Take the container and tray to the point of use.

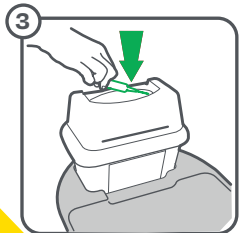

Waste Disposal

Dispose of sharps waste into the sharps container.
Please refer to page 13.

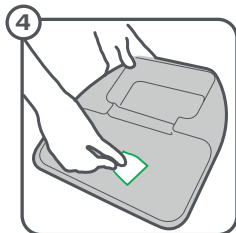

Reuse

Sanitise NPSD tray and ready for the next use.

Product Range Summary

Size	Product Code	Unit Dimensions mm	Carton Dimensions mm	Qty per Carton	Carton Weight kg	Carton Vol m ³
0.2L	5173-2430	146 x 79 x 37	300 x 395 x 320	100	6.4	0.040
0.45L	5170-2430	170 x 105 x 50	346 x 505 x 445	100	9.5	0.081
0.6L	5150-2439	224 x 105 x 54	455 x 450 x 530	100	12.75	0.108
1L	5160-2439	187 x 107 x 89	440 x 380 x 540	100	13.7	0.09
1.8L	5050-2430	220 x 125 x 114	412 x 364 x 600	75	12.05	0.089
2L	5140-5439	177 x 197 x 126	380 x 395 x 585	50	12.1	0.087
5L	5102-5430	260 x 170 x 205	399 x 490 x 593	48	16.7	0.1159
7L	5112-5430	342 x 170 x 205	397 x 628 x 595	48	20.4	0.1483
8L	36-4208	235 x 290 x 190	585 x 415 x 445	20	11.1	0.103
9L	5117-2430	291 x 292 x 195	530 x 383 x 585	20	13.8	0.119
12L	36-4212	310 x 290 x 190	585 x 410 x 495	20	11.6	0.118
13L	5115-2430	353 x 292 x 195	580 x 385 x 585	20	15	0.136
24L	5120-2439	335 x 450 x 230	749 x 454 x 454	15	16.5	0.159
25L XL	5195-1430	790 x 310 x 230	310 x 465 x 960	10	16.25	0.138
9L PA	5117-1430	291 x 292 x 195	530 x 383 x 585	20	14.6	0.119
13L PA	5115-1430	353 x 292 x 195	580 x 385 x 585	20	16	0.136
24L PA	5120-1430	335 x 450 x 230	749 x 454 x 454	10	17.7	0.159
25L Carton	2100-1001	370x 264 x 264	70 x 530 x 570	10	3.5	0.0255
50L Carton	2200-1100	579 x 293 x 293	70 x 790 x 585	10	2.15	0.0041
30L Clinisafe	5185-0000	375 x 400 x 300	825 x 425 x 640	10	16.1	0.224
60L Clinisafe	5186-0000	670 x 400 x 300	878 x 400 x 610	10	20.9	0.214

Accessories	Product Code	Compatible with	Qty per Carton
Wall Brackets and Trays			
Safe Clip Wall Bracket	4500-0000	1L, 1.8L, 2L, 5L, 7L Sharps Containers	10
NPSD 1, 1.8, 2 Litre Tray	4300-0000	1L, 1.8L, 2L, 3L Sharps Containers	10
0.45/ 0.6 Litre Wall Loop Bracket	4250-0000	0.45L, 0.6L Sharps Containers	10
9/ 13L Hook Bracket	SH-36-4115HB	9L, 9L PA, 13L, 13L PA Sharps Containers	50
9/ 13L Wall Bracket	SH-36-4115WB	9L, 9L PA, 13L, 13L PA Sharps Containers	50
Stands			
0.45/ 0.6 Litre Stand	4151-0000	0.45L, 0.6L Sharps Containers	-
1/ 1.8/ 2 Litre Stand	4205-0000	1L, 1.8L, 2L Sharps Container	26
5/ 7 Litre Stand	4257-0000	5,7L Sharps Container	-
9/13 Litre Stand	4207-0000	9L, 9L PA, 13L, 13L PA Sharps Containers	10
Cardboard Carton Plinth 25L	4410-0004	Clinisafe Carton 25L	-
Cardboard Carton Plinth 50L	4410-0005	Clinisafe Carton 50L	-
Trolleys			
9 / 13/ 24L Trolley	4480-0000	9L, 9L PA, 13L, 13L PA, 24L, 24L PA Sharps Containers	1

sharpsafe



CONTACT YOUR ACCOUNT MANAGER FOR INFORMATION & PRICING

Interpath Services
Leading the way

A Bunzl Company



SALES

+61 3 9056 5300

sales@interpath.com.au

www.interpath.com.au

FREE CALL

1800 626 369

